

Battery Charger

SB-205V

General

The SB-205V battery charger charges the SB-220V lithium ion battery.

Safety Information

WARNING A warning alerts the user to possible injury or death associated with the use or misuse of the instrument.

CAUTION A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Pay attention to all safety information in the Operator's Manual or Installation Guide.

WARNING

Never use the battery charger in the presence of any flammable anesthetic gas or high concentration oxygen atmosphere. Failure to follow this warning may cause explosion or fire.

WARNING

Never use the battery charger in a hyperbaric oxygen chamber. Failure to follow this warning may cause explosion or fire.

WARNING

Only use the provided power cord and connect it to a 3-pin AC outlet which is properly grounded. Otherwise, it may result in electrical shock or injury to the operator.

WARNING

Never charge a battery pack which is damaged, discolored or has leakage. It may cause overheating, explosion or fire.

WARNING

Only use the provided power cord. Using other power cords may result in electrical shock or injury to the operator.

CAUTION

Only charge a Nihon Kohden specified battery pack. If an unspecified battery pack is charged, it may cause overheating, explosion, fire or leakage.

CAUTION

Do not disassemble or modify the battery charger. It may cause overheating, fire, electrical shock or injury to the operator.

CAUTION

If the battery charger seems to be faulty, immediately disconnect the power cord from the AC outlet, attach a "Do not use" or "To be repaired" label on the battery charger and contact your Nihon Kohden representative.

CAUTION

For details on handling the battery pack, refer to the battery pack label, operator's manuals for the battery pack and device on which the battery pack is used.

CAUTION

Dispose of Nihon Kohden products according to your local laws and your facility's guidelines for waste disposal. Otherwise, it may affect the environment. If there is a possibility that the product may have been contaminated with infection, dispose of it as medical waste according to your local laws and your facility's guidelines for medical waste. Otherwise, it may cause infection.

CAUTION

Before cleaning or disinfecting the battery charger, disconnect the power cord from the AC outlet. Failure to follow this instruction may result in electrical shock and battery charger malfunction.

This Safety and Performance Information is an extract from the general and safety information sections of the most recent edition of Operator's Manual or Installation Guide. Therefore, the contents of your Operator's Manual or Installation Guide may differ from those of this Safety and Performance Information. For detailed operating procedures, follow the instructions of your Operator's Manual or Installation Guide.



Manufacturer

NIHON KOHDEN CORPORATION
1-31-4 Nishiocchiai, Shinjuku-ku,
Tokyo 161-8560, Japan
Phone +81 3-5996-8041
<https://www.nihonkohden.com/>



European Representative

NIHON KOHDEN EUROPE GmbH
Raiffeisenstrasse 10, D-61191 Rosbach, Germany
Phone +49 6003-827-0 Fax +49 6003-827-599

